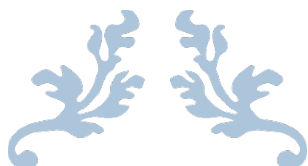
NightWatch

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
7. Technical Specifications

7.1 Specifications NightWatch

Essential Performance	[no essential performance]
Operating Mode	[Armmodule]: Body-worn
Weight of the Device	[Armmodule]: 35g
	[Basestation]: 90g
Dimensions (L x W x H)	[Armmodule]: 72 mm x 52 mm x 14 mm
	[Basestation]: 100 mm x 100 mm x 28 mm
Supply Voltage for the Charging Adapter	[Armmodule]: 100V – 240V AC, 50 Hz – 60 Hz
	[Basestation]: 100V – 240V AC, 50 Hz – 60 Hz
Current Consumption charging Adapter	[Armmodule]: 0,1A (RMS) max.
	[Basestation]: 0,2A (RMS) max.
Protection by enclosure	IP 20
Applied part classification	BF
Applied part	Armmodule
Admissible operating ambient temperature	15 °C - 35 °C
Admissible storage / transport ambient Temperature	-20 °C – 45 °C / -20 °C – 45 °C
Admissible relative humidity during operation	10 - 85 %RH
Admissible relative humidity during storage / Transport	10 - 90 %RH / 10 - 90 %RH
Battery	[Armmodule] Build in Lithium-ion battery 3.7V, 320mAh, 1.18Wh not replaceable 2 years if armmodule is off and is charged for 80%
Shelf Life	FCC ID: Y82-SC14S / IC: 9576A-SC14S
Armmodule contains	

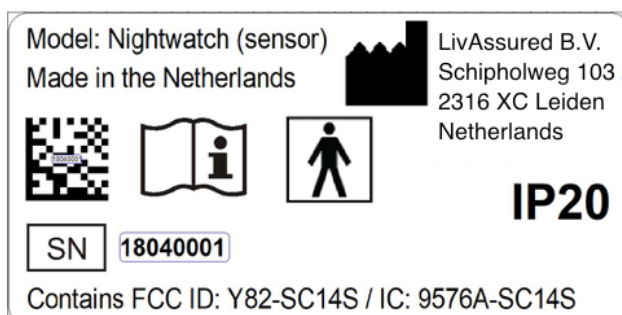
7.2 Product label

Armmodule label	
Date code: behind	2 characters yy= year 2 characters xx= week

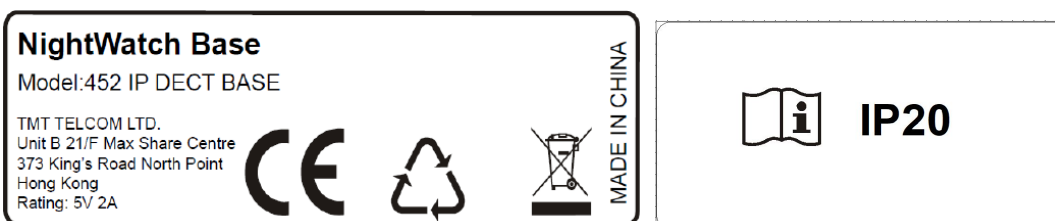
Armmodule back: with CE and WEEE symbol

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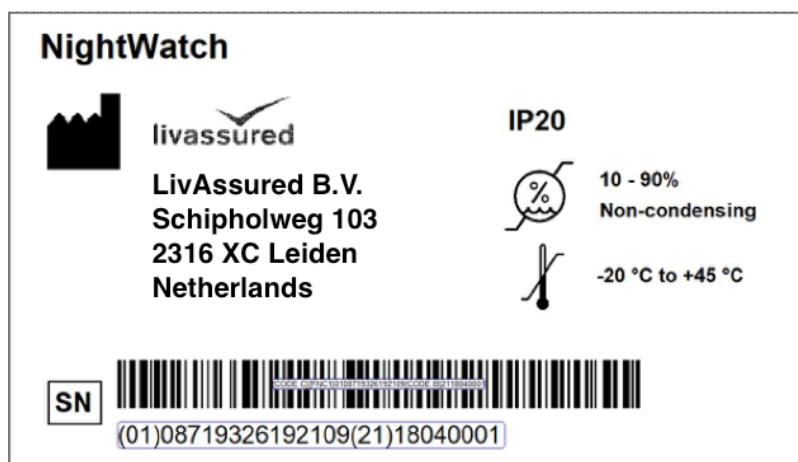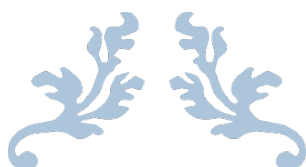
USER MANUAL

Contents

1.	Explanation of symbols used	53
1.1	Symbols in the user manual.....	53
1.2	Symbols on labels.....	53
2.	Safety instructions	54
2.1	General safety	54
2.2	Safety during use.....	55
2.3	Additional safety information for use at home	55
3.	The NightWatch	56
3.1	Product description and intended use.....	56
3.2	Medical classification.....	56
3.3	Intended users	56
3.4	NightWatch package contents.....	57
4.	The different components and how they work	58
4.1	Armband: arm module with flexible strap	58
4.2	Base station.....	59
5.	How to prepare your NightWatch for first use	60
5.1	Step 1: Adjust the armband.....	60
5.2	Step 2: First activation and charging the arm module	63
5.3	Step 3: Using the base station.....	64
5.4	Daily use	64
5.5	Transport.....	65
6.	Signals from the arm module and the base station	66
6.1	Arm module and base station display the same signal	66
6.2	Specific arm module signals.....	68
6.3	Specific base station signals.....	68
6.4	Adjusting base stations sound	69
6.5	Nurse call system	70
7.	Technical Specifications	71
7.1	NightWatch specifications	71
7.2	Product label	71
7.3	Packaging label.....	72
8.	Contact information.....	73
9.	Service life and guarantee	73
10.	Document revision.....	73

1.Explanation of symbols used

1.1 Symbols in the user manual

Warning

General mandatory precaution

1.2 Symbols on labels

The device contains electrical waste and must not be disposed of with normal household waste. After the usual service life, the NightWatch should be returned to the manufacturer. Please contact LivAssured for instructions: info@nightwatch.nl

Applied part, type BF

Manufacturer (including the manufacturer's address next to the symbol)

Serial number, production date followed by an identifier (yywwxxxx)

IP20

IP classification of protection by enclosure (No protection)

Consult IFU

CE Mark

Temperature limits (minimum + maximum)

Humidity limits (minimum + maximum)

2. Safety instructions

2.1 General safety

	- Only use this manual for product ID: NightWatch. - The instructions below must be followed. - Do not accept and use the device if there are signs of piercing, manipulation, water damage or any other damage to the packaging or label. - The NightWatch must only be repaired by qualified personnel. - Ensure that the labels on the product are present and legible at all times. Do not remove the labels. - Do not use this product in an environment where DECT signals may be blocked or interfered with by environmental properties or other equipment. - This product is intended to be used as an aid. This product does not guarantee that all epileptic seizures are detected and does not replace responsible and appropriate parental or professional supervision of the patient. - Always ensure the distance between the base station and armband does not exceed 10 metres.
	- Always fully charge the battery before use to guarantee uninterrupted use. - Check full functionality of the device before use. - Read the instructions for use before using the product.

2.2 Safety during use

	- Inspect all components for damage before use. - Please ensure the arm module, the base station, the various charging adapters and cables do not come into contact with water. - In case of damage to adapters, insulation materials or electrical parts, do not use the device and return to the manufacturer. - Do not wear the arm module while charging. - Never charge the arm module during wear. - Always ensure this product is used solely by people who can diagnose epileptic seizures and who know how to act in the event of an epileptic seizure.
	- Only use the charging adapter for the arm module as supplied by LivAssured to charge the arm module. - Only use the AC adapter as supplied by LivAssured to power the base station. - Discontinue use if skin irritation occurs and contact a medical professional.

2.3 Additional safety information for use at home

	- Do not use and place the NightWatch near a radiator. - Do not use the NightWatch in the bath or shower. - Do not submerge the NightWatch in water or other liquids.
---	---

3. The NightWatch

3.1 Product description and intended use

The NightWatch is intended to alert the user to possible night-time epileptic seizures. While the person with epilepsy (hereafter: the wearer) sleeps, carers can be notified when an 'urgent seizure' occurs. This refers to tonic-clonic, prolonged tonic or hypermotor seizures. The NightWatch continuously measures the wearer's movements and heartbeat. The device is intended to be used during sleep and will only display alerts when the wearer is in a recumbent position.

3.1.1 Essential performance

No essential performance has been determined. This product is intended to be used as an aid. This product does not guarantee that all epileptic seizures are detected and does not replace responsible and appropriate parental or professional supervision of the patient. Always ensure this product is used solely by people who can diagnose epileptic seizures and who know how to act in the event of an epileptic seizure.

3.2 Medical classification

The NightWatch is a class 1 device according to the 93/42/EEC directive.

3.3 Intended users

Patients with diagnosed epilepsy from 4 to 99 years of age with a light skin type.

PLEASE NOTE: This product should not be used for patients younger than 4 years of age.

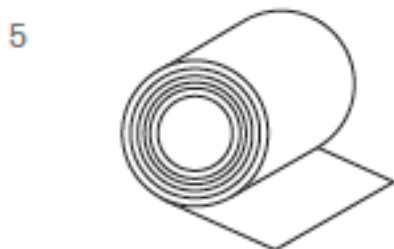
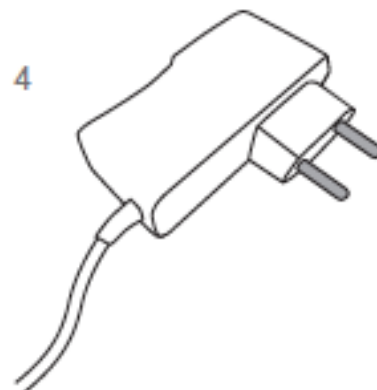
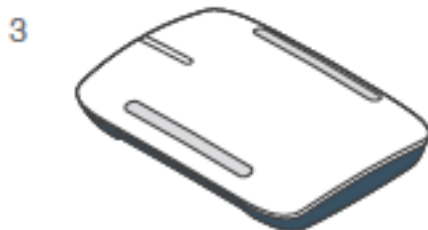
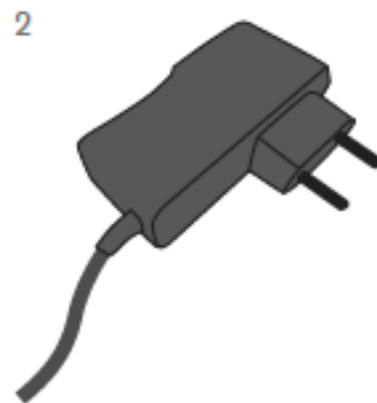
PLEASE NOTE: The use of the arm module on dark skin types could negatively affect the accurate determination of epileptic seizures.

3.4 NightWatch package contents


The NightWatch package contains the following items:

1. Base station
2. Power supply for base station (black)
3. Arm module
4. Charging adapter for arm module (white)
5. Flexible strap
6. Armband clips (4 in total, including 2 spare clips)

4. The different components and how they work

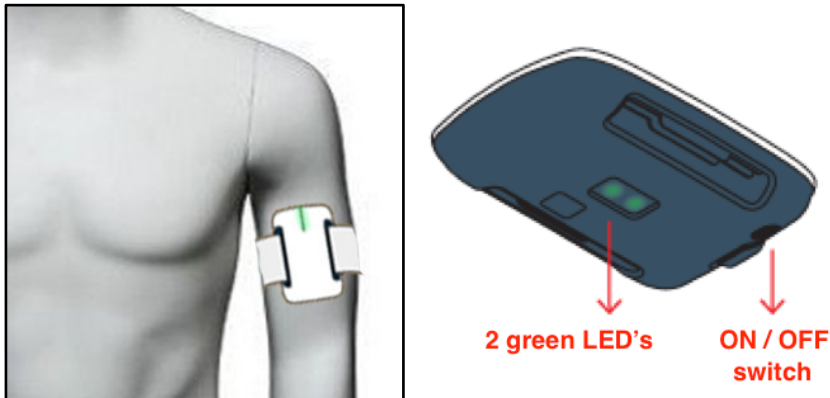4.1 Armband: arm module with flexible strap

The arm module is the device that is placed around the upper arm. The best position for the arm module is facing forward around the upper arm between the biceps and the shoulder.

The two green lights on the dark grey underside of the arm module are used for registering the heartbeat. The arm module also registers movement. It allows the arm module to detect an epileptic seizure and send a signal to the base station. The arm module is powered by a rechargeable battery.

The dark grey underside of the arm module has an on/off switch. The position can be changed using a pen, which should be done once at first use to switch on the arm module. The green heartbeat lights will light up. Then connect the armband to the charging adapter so that the system stops registering. As soon as you remove the armband from the charging adapter, the system will automatically start registering. This is how you start and stop the system from now on. Always keep the armband connected to the system when not using the system.

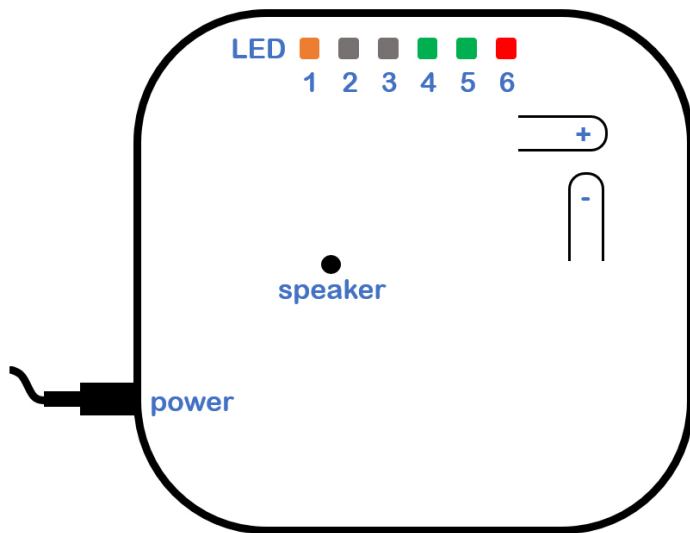
The arm module can only charge when turned on via the on/off switch. The arm module will not charge if the on-off switch is turned off. When it is not possible to connect the armband to the charging adapter, for example during transport, the on-off switch can be used to turn off the arm module to prevent the battery from running out.

PLEASE NOTE: The arm module cannot charge when turned off at the switch.

The white upper side of the arm module contains one red light and one green light, used for several signals.

4.2 Base station

The square white box is the base station that displays both light (LEDs) and sound signals. The base station can be placed anywhere in the home near a power outlet. However, ensure that you can hear the sound signal from the base station and that it is loud enough to wake you up at night.

5. How to prepare your NightWatch for first use

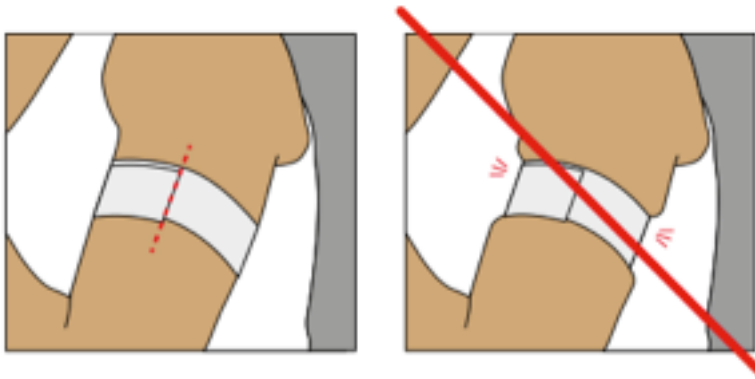
5.1 Step 1: Adjust the armband

You will need:

- Flexible strap
- 2 clips
- Arm module
- Ballpoint pen/pencil
- Sharp (fabric) scissors

The armband, consisting of the flexible strap and the arm module, should be adjusted to fit the wearer. The armband must not be too tight, but should fit snugly against the skin.

Use the flexible strap to measure the circumference of the wearer's arm above the biceps in a bent position. Do not stretch the band. Mark the flexible band at the overlap.

Cut the flexible strap at the marked point.

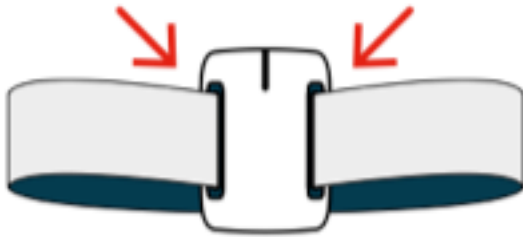
Attach the two clips to the band as shown below.

Tip: push one corner of the flexible band through the clip and then pull the entire band through the clip until it sticks out at a length of one centimetre.

Attach the clips to the slots in the arm module in a downward motion with the ends of the band protruding from the lower end of the arm module.

Place the armband around the wearer's upper arm, while making sure the arm module is facing forward, a little above the belly of the biceps but below the shoulder. This ensures the wearer is unlikely to lie on the arm module when turning onto his/her side.

PLEASE NOTE: In order to achieve an assessment of epileptic seizures that is as accurate as possible, the arm module should be worn on the upper arm (biceps) and directly on the skin. If the wearer is wearing a pyjama top or other long-sleeved garment, please ensure the sleeve is wide enough to wear the arm module underneath.

Check the tightness of the band. The armband will probably be a little too loose. The correct fit allows one finger to just about fit under the strap.

If the flexible strap feels too loose:

Remove the armband, pull the flexible strap through the clips further. Try again.

If the flexible strap feels too tight:

1. Using a fingernail, press the clips from below to remove them from the arm module.
2. Adjust the clips to make the flexible strap longer.
3. Place the clips in the slots again.

Try again.

Adjust the strap until it is both comfortable for the wearer and fits snugly around the arm. Finally, cut the excess ends off the strap so that they do not cover the green heartbeat sensors.

There is no cause for concern if the armband leaves an imprint on the arm after a night's sleep, as long as this imprint fades by itself within half an hour.

PLEASE NOTE: Please ensure the armband has been adjusted to the correct size for the patient before use, so that it is not too tight or too loose.

PLEASE NOTE: The flexible strap may protrude slightly on both sides of the dark underside. However, ensure that the ends do not cover the green light-up sensors. The band can be made tighter by pulling the ends. The band is very soft and the wearer will not experience discomfort from the protruding ends.

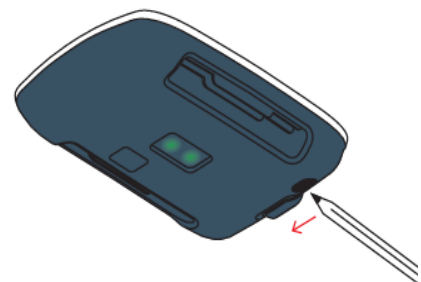
PLEASE NOTE: The flexible strap supplied measures 1 metre. This can be used to make a new armband when the wearer outgrows the first strap.

5.2 Step 2: First activation and charging the arm module

In principle, you only switch the arm module on once, after which the arm module is always on and can be charged. From now on, you will start and stop the system by connecting and disconnecting the arm module and the charging adapter.

- Take a ballpoint pen/pencil
- The dark grey underside of the arm module houses an on-off switch. Use the pencil to move this switch to the middle of the module to activate the arm module. This only needs to be done once. After this, you (de)activate the system via the charging cable. If you wish to turn the arm module off to prevent the battery from running out, use the on-off switch to turn off the arm module. PLEASE NOTE: the arm module will not charge if it is turned off.
- As soon as you have activated the arm module via the switch, 2 green LEDs will light up brightly on the dark underside. These are the LEDs that measure the heartbeat.
- Connect the arm module to the charging adapter and the power supply. The arm module will now charge.
- Then connect the base station to the power supply and wait until the arm module is charged.

Arm module battery life:

- Battery charging time: approximately 3 hours.
- In principle, a fully charged battery will last at least 12 hours.

Charging:

The arm module must be fully charged before each use. When you connect the armband to the charging adapter (which in turn is connected to the power supply), the heartbeat LEDs on the underside will turn off and a green LED on the white top part will start to blink. This indicates that the arm module is charging. As soon as the arm module is fully charged, the green LED on the upper side will be on continuously. The LED on the armband emits dim rather than bright light to minimise disturbance during the night.

PLEASE NOTE: Replace the product if a fully charged battery is empty after a single night or after one cycle of use.

PLEASE NOTE: Never allow the arm module battery to be completely empty, as this will make it 'lazy' and hold less charge in future.

PLEASE NOTE: Always keep the module connected to the charging adapter until use. This is because the module is activated as soon as it is removed from the charging adapter and will start to send alerts. It is not possible to 'overcharge' the arm module; there is no harm in leaving the arm module connected to the charging adapter.

5.3 Step 3: Using the base station

Ensure that the (activated) arm module is connected to the charging adapter. Position the base station where you want to receive the alert and ensure it is connected to a power supply.

After approximately 30 seconds, green LED 4 will light up continuously on the base station ('arm module connected to charging adapter') and green LED 5 will start to blink ('arm module battery is charging'). Once the arm module is fully charged, green LED 5 will also light up continuously.

Your system is now ready for use.

5.4 Daily use

- Roll up the wearer's pyjama sleeves, if necessary.
- Disconnect the armband from the charging adapter. The two green LEDs on the darker underside of the arm module will immediately light up brightly.
- Straight after disconnecting from the charging adapter, the base station will emit a warning signal indicating that no heartbeat has been detected yet.
- The red LED on the base station and the armband will blink red to indicate that no heartbeat has been detected yet.
- Place the armband around the upper arm, between the shoulder and the biceps, with the arm module facing forward on the upper arm so that the wearer will not lie on it when lying on his/her side.
- The base station will now blink green when the heartbeat is detected.
- Move the pyjama sleeve back down the arm if necessary.
- The wearer will now go to sleep. When the wearer is in a horizontal position and the arm module has not detected movement for 2 minutes, the more sensitive movement and heartbeat algorithms become active. When this occurs, the blinking green LED for the heartbeat changes to a continuous green LED.
- If the wearer rises or climbs out of bed, the more sensitive movement and heartbeat algorithms will be temporarily switched off, until the wearer is once again horizontal and at rest.

See 'Signals from the arm module and the base station' for all signals and what to do in case of issues.

PLEASE NOTE: Do not use water to clean the arm module and base station, please use an alcohol wipe. However, the flexible strap is flexible.

PLEASE NOTE: After disconnecting the armband from the charging adapter, a heartbeat warning signal will sound immediately until the heartbeat is found.

PLEASE NOTE: After use, connect the arm module to the charging adapter immediately. Do not switch off the arm module during charging or during use. Only switch off the arm module when returning it for service or during transport to a different location.

PLEASE NOTE: Please ensure the base station is never covered during use.

PLEASE NOTE: Please ensure that the base station is switched on before the arm module is removed from the charging adapter. In order to verify that the base station is working correctly, wait for the warning signal from the base station, indicating that the arm module was removed from the charging adapter and has not (yet) detected a heartbeat. After this, the arm module can be placed on the wearer's arm.

5.5 Transport

It may be that you wish to take the system to another location. When you remove the arm module from the charging adapter for transportation, the green heartbeat LEDs on the arm module will remain on. You should switch off the arm module to prevent the arm module battery from losing power during transportation. Turn off the arm module using the on-off switch on the dark underside, where you also turned the arm module on. Move the switch to the outside of the arm module using a pen/pencil. The green heartbeat LEDs on the underside of the arm module will now switch off. The arm module is now switched off and can be transported without the battery becoming empty.

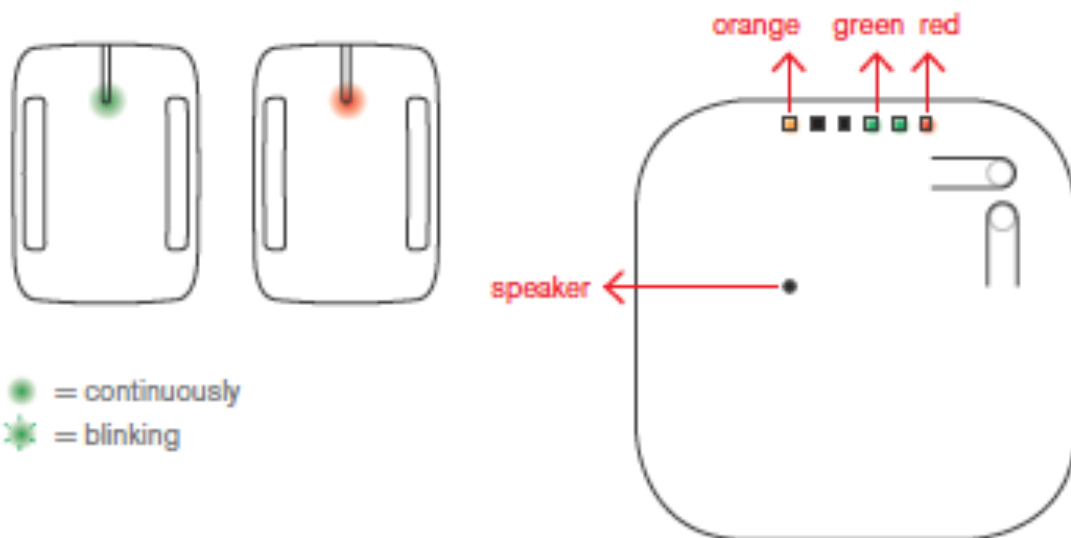
PLEASE NOTE: The arm module cannot be charged if it is switched off. The arm module must be switched on to charge. If you wish to use the system again, you can use the on-off switch to turn the arm module back on and charge it.

6. Signals from the arm module and the base station

Both the arm module and the base station display signals about the status of the system. The arm module uses a green LED and a red LED to display these signals. The base station uses two green LEDs and a red LED, and is also supported by sound signals. The signals used are shown below.

6.1 Arm module and base station display the same signal

 Continuous green LED

- The NightWatch is able to detect seizures. The heartbeat is measured reliably, the connection between arm module and base station is in order, the battery is in order, the wearer is in a horizontal position (is lying in bed), the armband wearer has been at rest for at least 2 minutes.

RING

Rapidly blinking red LED (and the base station emits a 'ringing' sound)

- A possible epileptic seizure has been detected! Check on the armband wearer.

BEEP

Repetitive 1x red LED and the base station emits 1 beep per 2 seconds.
This could mean one of two things:

- The arm module has been removed from the charging adapter, but no heartbeat has been detected yet.
This signal will stop as soon as the NightWatch detects the heartbeat after the armband has been put on the wearer. Lie down quietly for the heartbeat to be detected faster.
- No reliable heartbeat measured for more than 2 minutes.
This may occur e.g. when the armband is too loose, has slipped off the wearer's arm, or was not connected to the charging adapter after use. It is also possible that the wearer is lying on the armband, impeding detection. Adjust the wearer's armband (tighter) or connect it to the charging adapter.

2 BEEPS

2 BEEPS

2 BEEPS

Repetitive 2x red LED and the base station emits 2 beeps per 2 seconds.

There is no connection between the arm module and the base station. Several causes are possible:

- *The arm module is switched off. Turn on the arm module using a pen to move the on-off switch on the dark underside. The green LEDs on the underside of the arm module should light up when it is on and has been disconnected from the charging adapter.*
- *The arm module and the base station are too far apart. Place the base station and armband closer together.*
- *It may also be that only the arm module sends the light signal described above. In that case, check whether there is a power supply to the base station.*

4 BEEPS

4 BEEPS

4 BEEPS

Repetitive 4x red LED and the base station emits 4 beeps per 2 seconds.

- The battery of the arm module is empty.
You will not be able to use the NightWatch for this coming night. Connect the arm module to the power supply. If it is still not charged the next morning, the arm module is defective. Please contact us on info@nightwatch.nl.

6.2 Specific arm module signals

Continuous green LED

- Not connected to the charging adapter: NightWatch alerts are active. A heartbeat is registering, the connection with the base station is in order, the battery is in order, the wearer is at rest and in a horizontal position.
- Connected to the charging adapter: the arm module is fully charged; the battery is in order.

Blinking green LED:

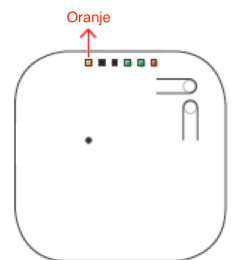
- Not connected to the charging adapter: the arm module has detected a heartbeat, but the armband wearer is not at rest yet. The NightWatch alerts are not active yet. The NightWatch will become active as soon as the wearer is at rest in a horizontal position for more than two minutes.
- Connected to the charging adapter: arm module is still charging.

6.3 Specific base station signals


Continuous Orange LED

- Base station is connected to power supply.


One continuous green LED and one blinking green LED

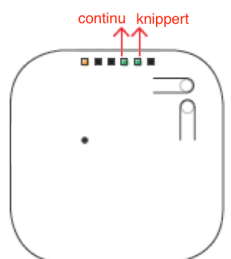- The arm module is connected to the charging adapter and still charging.

Two continuous green LEDs

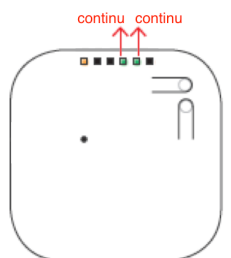
- The arm module is connected to the charging adapter and is fully charged.


Blinking green LED:

- The arm module has detected a heartbeat, but the armband wearer is not at rest yet.

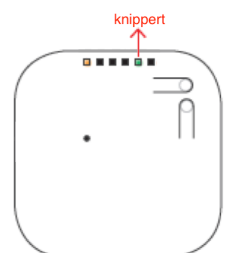The heartbeat is being measured, but the NightWatch is not yet fully active for alerts. As soon as the wearer is at rest in a horizontal position for more than two minutes, the NightWatch alerts will be fully active.

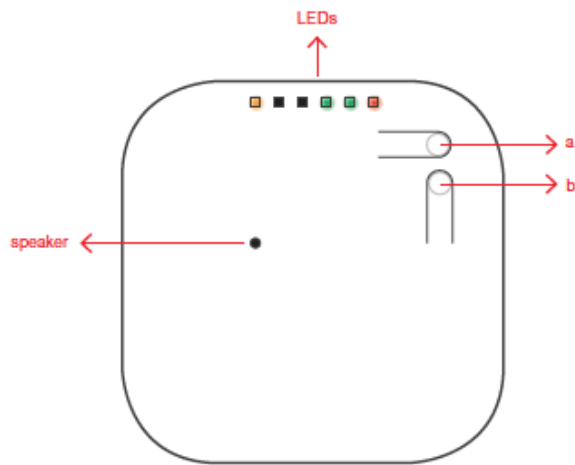6.4 Adjusting base station sound

The 'a' and 'b' buttons

These buttons allow you to do the following:

- Adjust the volume
 - Both buttons can be used to adjust the alert volume. Press down button 'a' or 'b'. You will hear the alert sound become more (a) or less (b) loud. Release the button at the desired volume. The volume is now set and will be saved when you disconnect the base station from the power supply. You can adjust the volume as you wish.
- Temporary pause the sound
 - One of the buttons (it doesn't matter which one) can be pressed if you have heard an alert and wish to turn it off. This is temporary. A subsequent alert will be audible again.

PLEASE NOTE: Ensure that the base station alert signal is set to an audible sound level during the entire cycle of use.

PLEASE NOTE: Loud noises from the base station can possibly cause epileptic seizures. Ensure that the sound signals are only loudly audible to the carer, not the wearer.

6.5 Nurse call system

The NightWatch can be linked with a Nurse Call system. This should be installed and configured in consultation with the Nurse Call supplier and the NightWatch supplier. Please contact the LivAssured helpdesk.

Always ensure this product is used solely by people who can diagnose epileptic seizures and who know how to act in the event of an epileptic seizure.


This product is intended to be used as an aid. This product does not guarantee that all epileptic seizures are detected and does not replace responsible and appropriate parental or professional supervision of the patient.

7. Technical Specifications

7.1 NightWatch specifications

Essential Performance	[no essential performance]
Operating Mode	[Arm module]: Body-worn
Weight of the Device	[Arm module]: 35g
	[Base station]: 90g
Dimensions (L x W x H)	[Arm module]: 72 mm x 52 mm x 14 mm
	[Base station]: 100 mm x 100 mm x 28 mm
Supply Voltage for the Charging adapter	[Arm module]: 100V – 240V AC, 50 Hz – 60 Hz
	[Base station]: 100V – 240V AC, 50 Hz – 60 Hz
Current Consumption Charging Adapter	[Arm module]: 0.1A (RMS) max.
	[Base station]: 0.2A (RMS) max.
Protection by enclosure	IP 20
Applied part classification	BF
Applied part	Arm module
Admissible operating ambient temperature	15 °C - 35 °C
Admissible storage / transport ambient Temperature	-20 °C – 45 °C / -20 °C – 45 °C
Admissible relative humidity during operation	10 - 85 %RH
Admissible relative humidity during storage / Transport	10 - 90 %RH / 10 - 90 %RH
Battery	[Arm module] Built-in Lithium-ion battery 3.7V, 320mAh, 1.18Wh not replaceable
Shelf Life	2 years if arm module is off and is charged for 80%
Arm module contains	FCC ID: Y82-SC14S / IC: 9576A-SC14S

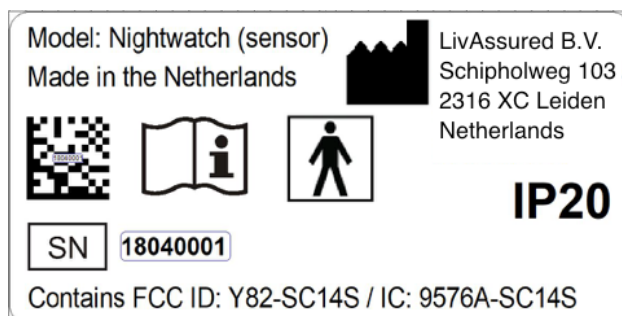7.2 Product label

Arm module label
 Date code: behind 2 characters yy= year
 2 characters xx= week

Arm module back: with CE and WEEE symbol

NightWatch Base
Model: 452 IP DECT BASE
TMT TELCOM LTD.
Unit B 21/F Max Share Centre
373 King's Road North Point
Hong Kong
Rating: 5V 2A

CE

MADE IN CHINA

IP20

Base station labels

7.3 Packaging label

NightWatch

livassured

LivAssured B.V.
Schipholweg 103
2316 XC Leiden
Netherlands

IP20

10 - 90%
Non-condensing

-20 °C to +45 °C

SN **(01)08719326192109(21)18040001**

Date code after (21): first 2 characters yy= year
next 2 characters xx=week

8. Contact information

Manufacturer	Customer Service
LivAssured Venusstraat 17 4105 JH Culemborg Netherlands Email: info@nightwatch.nl Website: www.nightwatch.nl	LivAssured Schipholweg 103 2316 XC Leiden Netherlands Email: info@nightwatch.nl Website: www.nightwatch.nl

9. Service life and guarantee


The NightWatch has a 2-year guarantee. In the event the NightWatch is not working or seems to be working incorrectly, please contact LivAssured via email: info@nightwatch.nl

The expected service life of the NightWatch in case of normal use is a minimum of 5 years. At the end of its service life, return the NightWatch to LivAssured. Contact info@nightwatch.nl for instructions.

10. Document revision

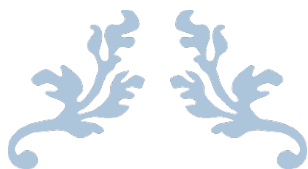Rev.	Date	Author	Description of changed
1.0	23-October-2017	A.J. de Blaeij	First version
1.1	29-November-2017	A.J. de Blaeij	Amendments as a result of product safety study
1.2	22-January-2018	A.J. de Blaeij	labels were amended and rev overview was added
2.0	19-February-2018	A.J. de Blaeij	Logo was added + document pages changed to multiples of 4 due to printer
2.1	17-September-2018	M.R.P. Jansen	Improvement to Dutch + clarification on positioning the armband + various other clarifications
2.2	24-January-2019	M.R.P. Jansen	Warnings amended
3.0	21-March-2019	M.R.P. Jansen	Multiple languages added

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